



SUPPLY MENU



UTM® UNIVERSAL TRANSPORT MEDIUM™

FLUID | RPP

Used for Nasal collection.



PP FECAL CONTAINERS

GI



QUANTISAL™ ORAL FLUID COLLECTION DEVICE

Toxicology Oral Fluids



HYDRAFLOCK® SPECIMEN COLLECTION SWAB

UTI | Vaginitis | PGX | Fungal

UTI (Place swab into 15ml tube)

Vaginitis (Place swab into 15ml tube)

PGX (For PGX place back into original paper packaging.

GI (well covered swab and placed in 15ml tube)



15 ML CENTRIFUGE TUBES

UTI | Vaginitis | Fungal | GI | Nail

This tube is used to transport specimen for UTI swabs, vaginitis swabs, Fungal swabs, and stool samples



PODIATRIST NAIL CLIPPERS

Nail



SUPPLY MENU



URINE CUP WITH INTEGRATED TRANSFER DEVICE

Toxicology | Urinalysis | UTI | STI

This is a urine collection and transfer device. Please do NOT send this cup to the lab, it will leak. Use this device to transfer urine into one of the tubes. Send ONLY the tubes.



VACUETTE URINE COLLECTION TUBES

Urinalysis

To be used with Urine Cup with Integrated Transfer Device.
Do not send the cup, only send the tube.



VACUTAINER® PLUS C & S PRESERVATIVE

UTI | STI

To be used with Urine Cup with Integrated Transfer Device.
Do not send the cup, only send the tube.



SPECIMEN BAG

Place each patient sample in an individual specimen bag. Any paper work can go into the paperwork pouch on the outside of the bag. The specimen goes into the sealed portion of the bag. Do not place paperwork inside the bag with the specimen.